Product Baseline: TomoTherapy

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



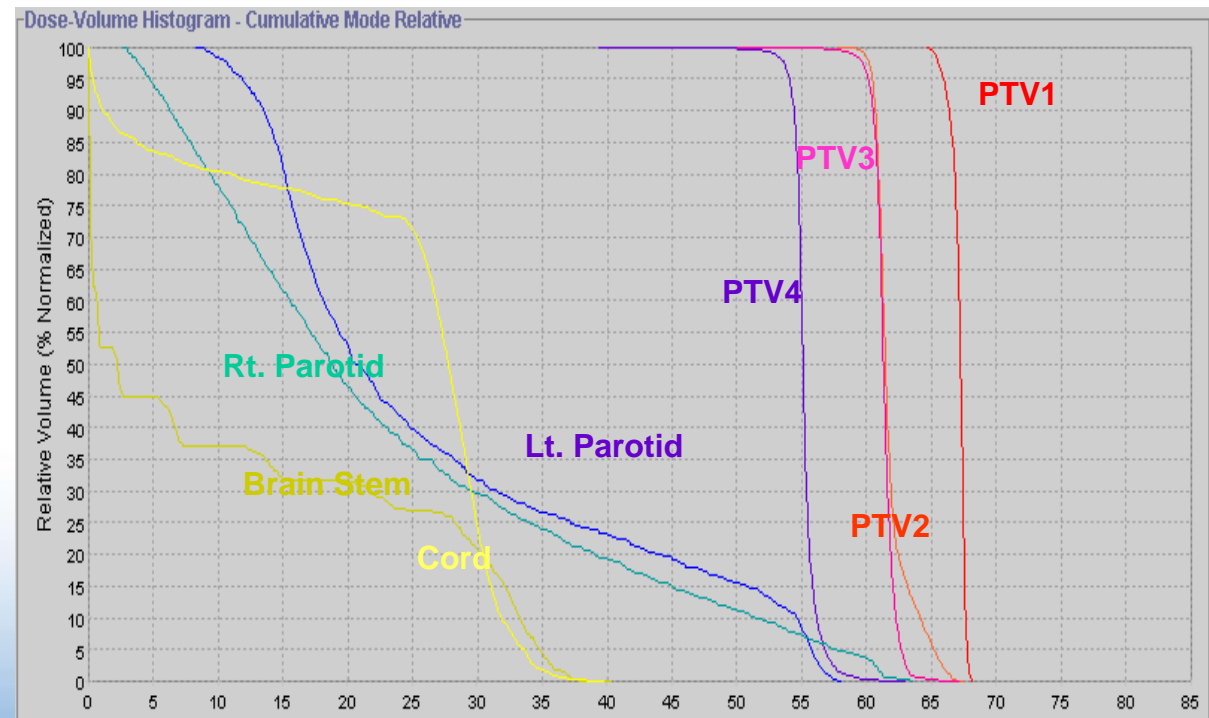
Small Treatment Volume

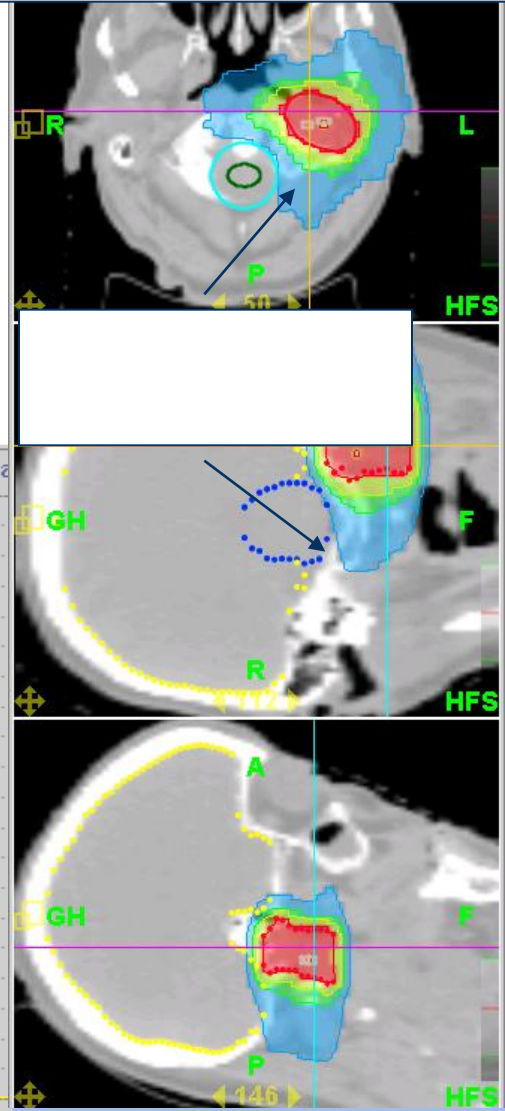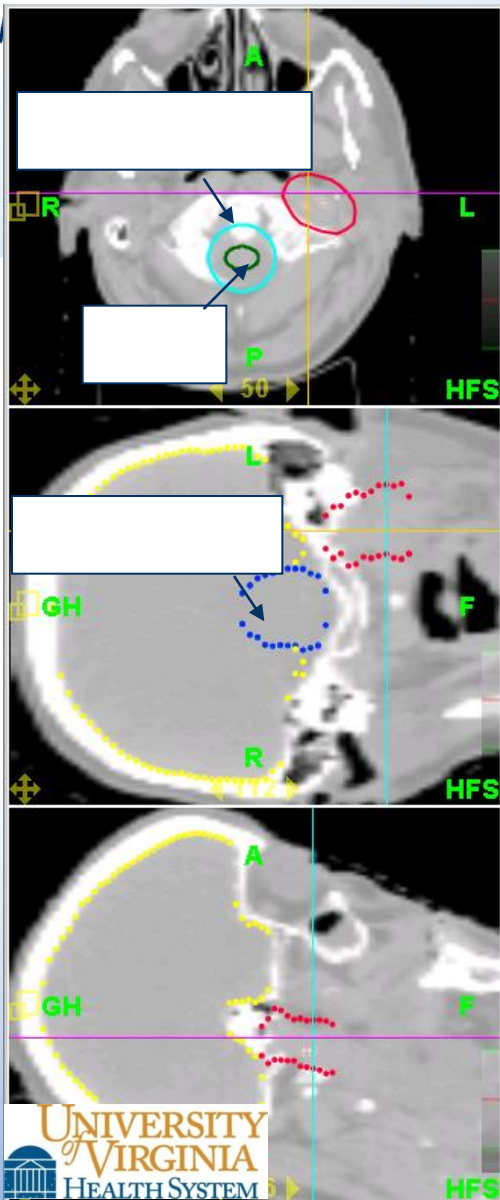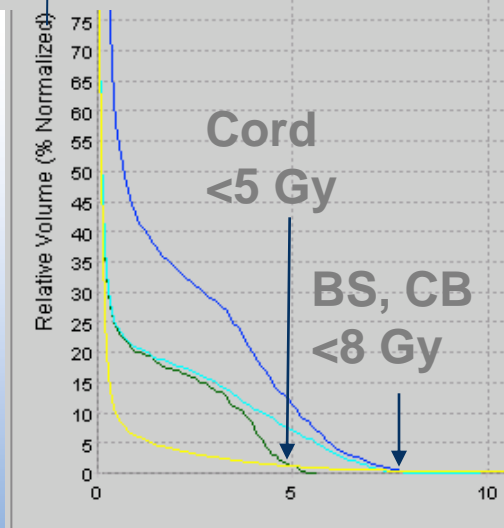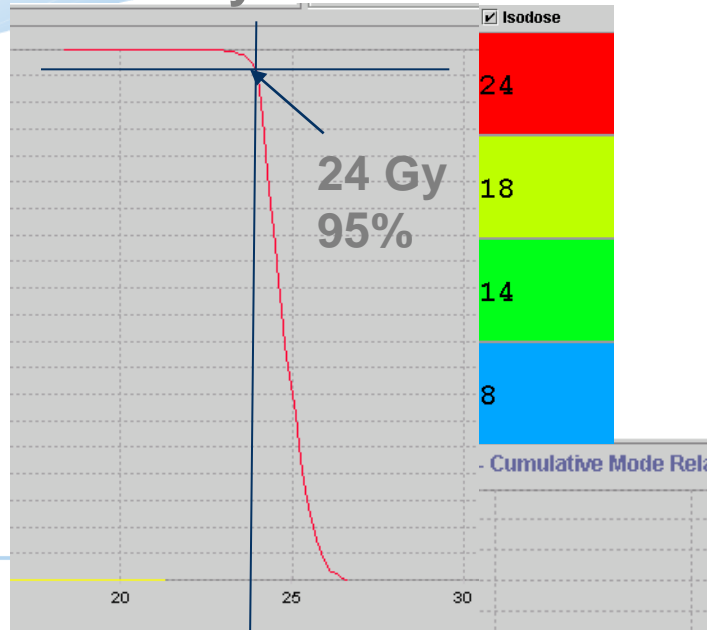4 X 6 Gy Fractions

TomoDirect for Simple Head and Neck Case

Planning Station

Patient: [Redacted] Plan Label: **Plan_02** User Name: morning warm up
 No Photo DOB: [Redacted] Plan Status: **Approved**
 Sex: [Redacted] Plan Date: **Oct 22, 2009 5:20:01 PM**
 ID: [Redacted] Position: **HFS**
 Oncologist: [Redacted]
 Disease: **10936**

What's Next
Plan Approved
 Click **Generate Plan Report** to create a plan report.
 You may now perform **Delivery Quality Assurance** to verify the planned dose.

Save

Contouring ROIs Plan Settings Beam Angles Calculation Fractionation

Angles

Name	Display	Angle (degrees)	-Xg (leaves)	+Xg (leaves)	Status
Angle	<input checked="" type="checkbox"/>	0.0	None	None	Applied
Opposed	<input checked="" type="checkbox"/>	204.8	None	None	Applied
Angle1	<input checked="" type="checkbox"/>	326.9	None	None	Applied

Reset Angles Protocols

Apply

Display Options
 Lines and Wash Lines Only Wash Only
 View Entire Treatment

Targets

Name	Color	Display	Use Angle
PTV	[Red]	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>

Regions at Risk

Name	Color	Display
External	[Blue]	<input type="checkbox"/>
CT MARK	[Green]	<input checked="" type="checkbox"/>
Mandible	[Yellow]	<input checked="" type="checkbox"/>
GTV	[Red]	<input checked="" type="checkbox"/>
cord	[Green]	<input checked="" type="checkbox"/>
trachea	[Purple]	<input checked="" type="checkbox"/>
esophagus	[Brown]	<input checked="" type="checkbox"/>

Transverse

Coronal

Sagittal

61

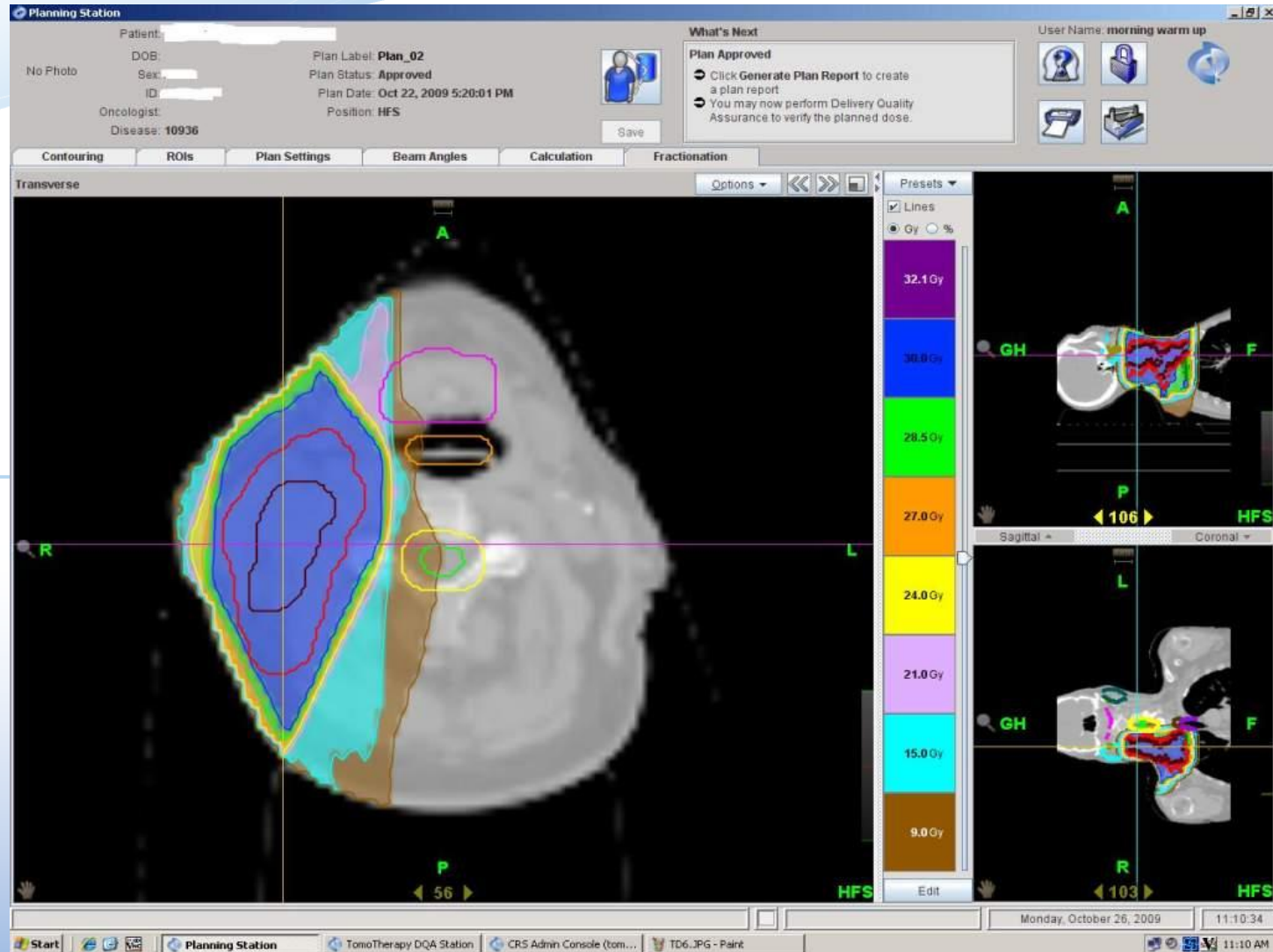
129

11:06:38



Monday, October 26, 2009

Start Planning Station TomoTherapy DQA Station CRS Admin Console (tom...)

TomoDirect for Simple Head and Neck Case

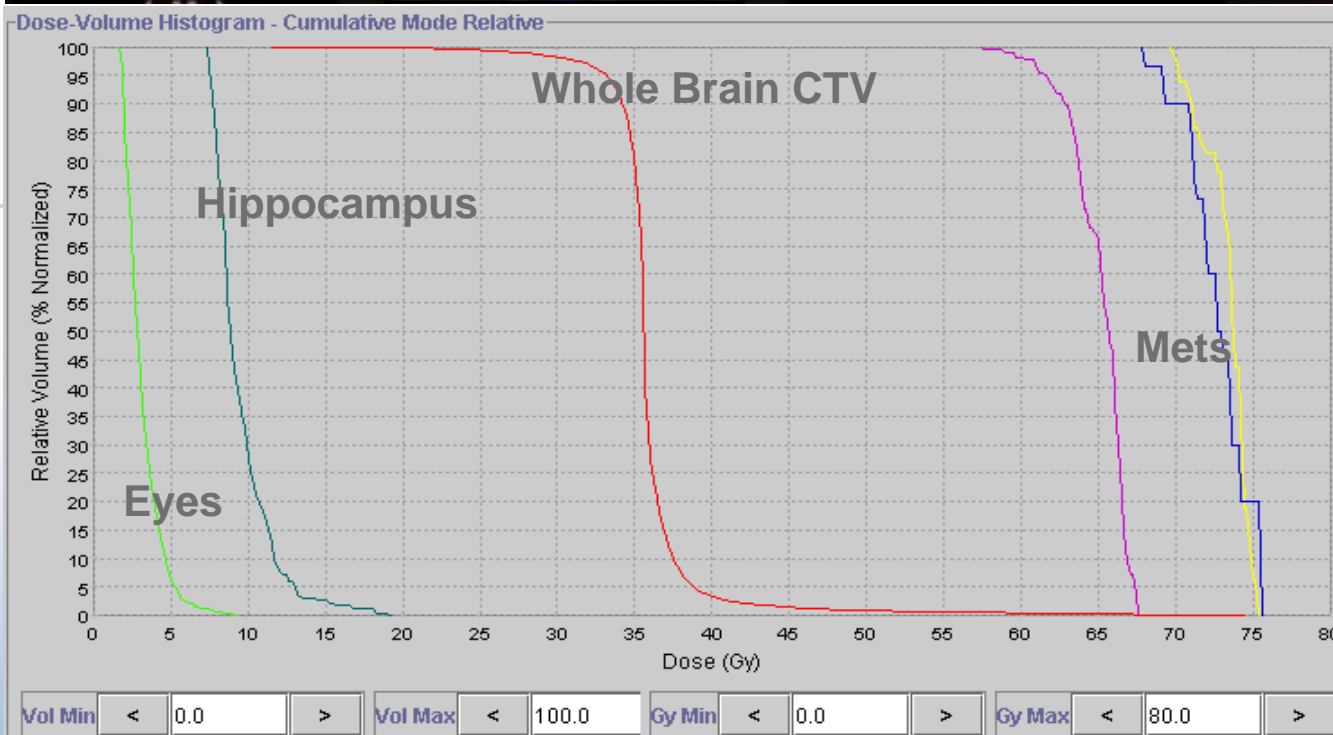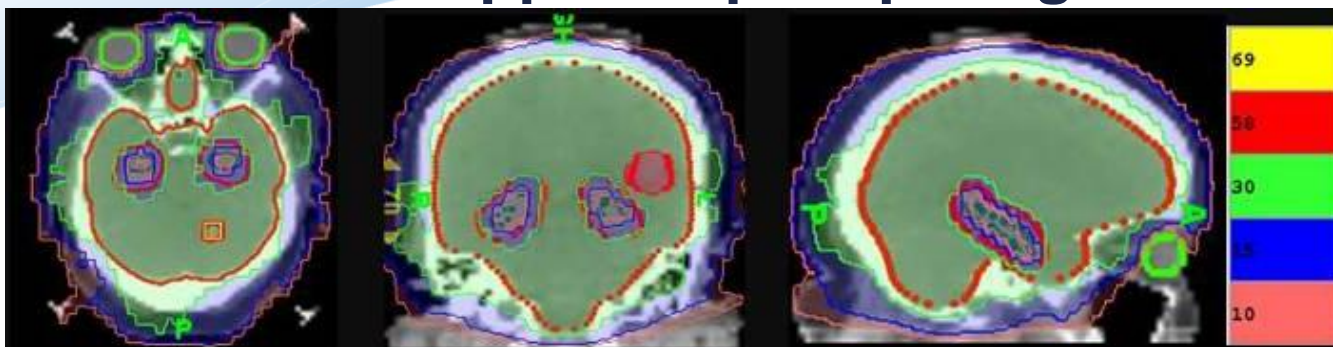Whole Brain + Simultaneous In-field Boost with Hippocampal Sparing

Whole Brain + Simultaneous In-field Boost with Hippocampal Sparing

American Journal of Clinical Oncology • Volume 30,
Number 1, February 2007

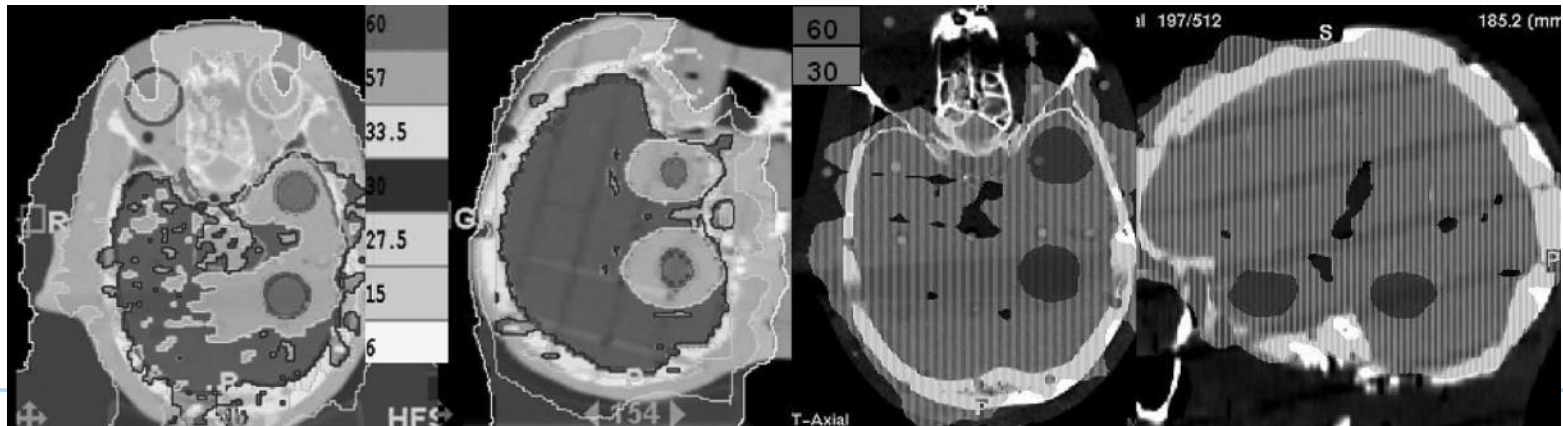
FIGURE 1. Helical tomotherapy for a phantom case with 3 lesions where 2 lesions are located in the same plane. Despite the coplanar delivery inherent in helical tomotherapy, good sparing of brainstem, optic chiasm, and eyes is maintained.

CONCLUSION

We have confirmed the feasibility of using helical tomotherapy to deliver a radiosurgery type boost synchronously with WBXRT in planning and phantom studies. We are examining the clinical feasibility of this approach in a phase I clinical trial currently in progress where the target dose is 30 Gy WBXRT (over 10 fractions) with a concurrent boost to individual lesions with an additional 30 Gy to a total dose of 60 Gy (in 10 fractions).

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